

REPAIR KIT INSTRUCTION SHEET

YOUR BREASTFORM MUST BE DRY:

- **Ensure there are NO air bubbles in the silicone of the form OR under the skin of the form near the tear.**
- Cut a CIRCULAR patch approx 3 times as wide as the damaged area. ie if the hole is about the size of a 5c piece the make the patch about the size of a 20c piece.
- **To remove air bubbles GENTLY push at the sides of the tear until the air bubble is forced out.**
- Clean around the damaged area with a skin tac wipe..
- Peel off the protective backing of the repair tape and apply to the damaged area. Be careful not to trap air bubbles under tape.
- Use the back of your fingernail to gently rub around the damaged area FROM THE CENTRE OUTWARDS - making sure there are NO CREASES or AIR BUBBLES under the tape. Do NOT rub over the hole it self – only around it as closely as possible.
- Place the breastform back into its protective plastic insert in the box it was shipped in. Leave for 2-3 days.
- If the damage is to the back of the form you may need to leave it lay flat on a hard surface in stead.

IF a mistake is made you must start over. GENTLY remove the tape from the form. Results may vary and repairs to seams may not prove effective. Repairs may last weeks or months. Repairs can be repeated depending on the nature of the damage.

To avoid future damage – WEAR Cotton GLOVES while putting on your forms.

